



Mercury-free sphygmomanometer

KM-380II

New model of Mercury-free sphygmomanometer

Turn on “**quickly**” and display
appears “**quickly!**”



KENZMEDICO
ORIGINAL

RADICAL MOVEMENT
Quick stand-by

NEW BACKLIGHT
New backlight LCD



Washable cuff



KENZMEDICO

Easier to use and more convenient Mercury-free sphygmomanometer.
When pressing the power button, it turns on “quickly” and the display appears “quickly” with simple operation!



KENZMEDICO ORIGINAL

Quick stand-by

- ✓ **Shorten the time for Stand-by.**
Active in 2 seconds after turning ON the power.

New backlight LCD

- ✓ **The 2-stage illuminance function is integrated.**
•Illuminance will be lowered in 10 seconds without any operation, then turn off the light in 50 seconds without any operation after illuminance lowering.
•Lighting constantly during measurement.
- ✓ **Prevention from incorrect operation during measurement.**
Even if you press the power button erroneously during measurement, the power will not turn off.

- ✓ **Automatic power off function integrated**

It turns OFF automatically without any operation more than 5 minutes.

- ✓ **Battery life**
Approx. 6,000 measurements available

- ✓ **Light-weight design**

- ✓ **Slip resistance sole**

- ✓ **Non-latex specification**
Made from Non-Latex materials for bulb, bladder and tube.
The healthcare professionals and patients with latex allergy can use it safely.



Visible in a dark place



Washable cuff

- Non-water-permeable and non-bladder one-piece cuff. (Urethane coating on the inner surface)
- Easy cleaning with disinfectant. Cuff is washable when tube is removed and the connector plug is attached.



Product specification

Trade name	Mercury-free sphygmomanometer KM-380II
Pressure range	20 to 300 mmHg in 2-mmHg scale
Accuracy of intracuff pressure	Within ±3 mmHg
Dimension (mm)	In housing: Approx. 334(depth)×Approx. 96(width)×Approx. 65(height) In use: Approx. 334(depth)×Approx. 96(width)×Approx. 338(height)
Weight of main body	Approx. 680g (with battery and without cuff and squeeze bulb)
Power protection	Internally powered equipment, applied part of BF type *This product conforms to EMC standard IEC 60601-1-2
Electric power source	3×AA batteries (rechargeable battery available) *Measurable about 6,000 times in using size AA batteries
Automatic power OFF	Turning off automatically without any operation more than 5 minutes.
Electric rating	DC4.5V
Power consumption	Max 0.2W (with backlight on)

*The product specification may be changed without prior notice.

KENZMEDICO CO., LTD.

Head office:
552-1, Kyohei Kodamacho, Honjo-city, Saitama, Japan 367-0206
TEL: +81-495-71-1001 FAX: +81-495-72-6716
Website: <http://www.kenzmedico.co.jp>