## HEINE EL3 LED Examination Light

# HEINE EL3 LED Examination Light

DEUTSCH	3
ENGLISH	5
FRANÇAIS	7
ESPAÑOL	9
ITALIANO	11
SVENSKA	13
NEDERLANDS	15
DANSK	17
NORSK	19
SUOMI	21
PORTUGUÊS	23

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# HEINE EL3 LED Examination Light

 Please read and follow these instructions for use of and keep them for future reference.

## Intended Use

The EL3 LED is a mains operated medical examination light for illuminating the body's surfaces and cavities during medical examinations in closed examination rooms. It should only be operated by qualified medical personnel within a professional healthcare facility.

 The device is intended only for medical examinations in which failure of the light will not pose a risk to the patient. A main source of lighting must always be switched on.

## For U.S. only:

 **Federal law restricts this device to sale by or on the order of a Physician or Practitioner.**

## Warnings and Safety Information

 **CAUTION!** Indicates potential hazardous situations. Ignoring the corresponding instructions may lead to dangerous situations of mild to moderate extent. (Background color yellow; foreground color black).

 **NOTE!** This symbol indicates valuable advice in terms of set up, operation or maintenance, as applicable. Notes are important, but not related to hazardous situations.

## Product overview

### Gooseneck:


- 1 Gooseneck (flexible component)
- 2 Gooseneck (rigid component)
- 3 Gooseneck (flexible component)

### Lamp head:

- 4 Hand grip

### Lamp base:

- 5 ON/OFF switch (system)
- 6 Connection socket for the power cable
- 7 Installation cover
- 8 Connection for the mounting systems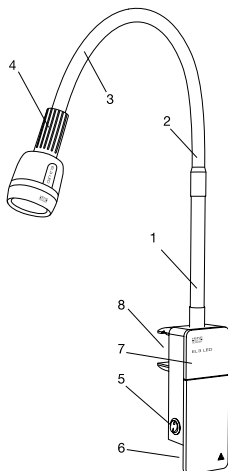

## Installation instructions

When mounting the accessories, please follow the enclosed mounting instructions for the mounting systems.

 To avoid risk of electric shock, this equipment must only be connected to a supply mains with protective earth.

To ensure all-pole disconnection from the mains at all times, the ME device must be installed so that the power cord is accessible and disconnectable.


When using the examination light with a stand, clamp or wall mount, ensure the examination light is standing or stably attached. Check that the attached examination light is stable and secure before each use.

Use the accessories supplied with the examination light to ensure the examination light is held securely and properly in the respective mount.

When moving the lamp, ensure that the power cable is not taut. Unplug the power cable when moving the lamp over longer distances.


When mounting the lamp to the wall, ensure that the subsurface is able to bear the mechanical stress. The wall plugs supplied with the product are universal wall plugs and suitable for most building materials (e.g. concrete, solid brick, plasterboard or gypsum cardboard). Before mounting the device, check whether you need to use a special wall plug for the subsurface of your wall.

Before mounting the device to the wall, check that there are no cables or pipes running under the planned drill holes that could get damaged.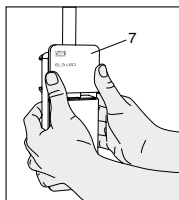

For mounting the mains adaptor to the wall, clamp or stand mount, push up the cover (7) and take it off. Use the screws supplied with the product to fasten the examination light to the respective mounting system (clamp or stand mount) or use the wall plugs supplied to mount the lamp to the wall.

The screws for the mounting systems are located in the corners of the mounting unit.

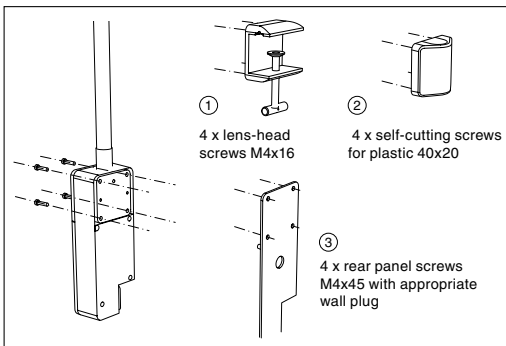### ① Clamp mount:

Please use the screws enclosed with the product to attach the clamp mount to the base of the lamp. This can be fastened to an even surface (e.g. table top) with a maximum thickness of 40 mm or to a circular tube with a maximum diameter of 30 mm. Please check that the mounted lamp is securely in place before using the lamp.

### ② Stand mount:

This mount system is designed for tubes with a diameter of 25 mm. Please use the screws enclosed with the product to attach the stand mount to the base of the lamp.

### ③ Wall mounting:

The rubber mat can be used as a drilling template for marking the holes on the wall and is intended to level out uneven areas on the wall for the mains adaptor. Please use the screws in combination with the wall plugs supplied with the product when mounting the device to a suitable wall.

## Setting up

 The EL3 LED must not be used for eye examinations or as a surgical light.

The EL3 LED is intended for use in closed spaces. It is not designed for mobile use or for use outdoors. Improper use can result in damage to the device or pose a risk to the user.

Prior to each use, check that the EL3 LED is functioning properly as intended. Do not use the device if visible damage is apparent.

The device does not have an application part and therefore care must be taken to ensure that the examination light does not come into contact with the patient!

The EL3 LED must not be used in strong magnetic fields such as within the vicinity of MRI equipment.

The EL3 LED must not be used in oxygen-enriched environments.

The EL3 LED must not be used in an environment of flammable anaesthetic agents (class AP) or with oxidising agents (class APG).

Keep a minimum distance of 20 cm between the examination light and the examined object (patient).

Do not cover the examination light.


Ensure that the device is handled properly and carefully.

Do not let the power cable become taut as this could damage the device or pose a tripping hazard.

Only after the device has been securely mounted, should the power cable supplied with the device be connected by plugging it into the mains connection socket (6). To use the device, switch it on by pressing the power switch (5) so that the switch is in the ON position I and the switch lights up green.

## Positioning

- ⚠ To avoid contamination, position the examination light prior to the examination or by an assistant during the examination.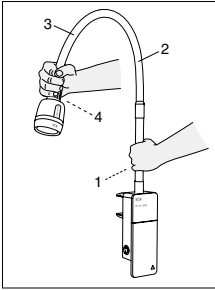

The swivel arm of the EL3 LED examination light has two flexible (1, 3) and one rigid section (2).

To position the light precisely, hold the light head by the hand grip (4) and the gooseneck by the flexible part (1) and bend the gooseneck into the desired position. Fine adjustments can be made via the hand grip (4).

Do not bend the gooseneck into a <math><90^\circ</math> degree angle.

The picture shows the positioning of the lamp during transport on a stand.

## Operation

- ⚠ Keep a minimum distance of 20 cm between the examination light and the examined object (patient). The optimal working distance is 30cm.

## Hygienic reprocessing

The instruction is available:

- at [www.heine.com](http://www.heine.com)
- in a paper version which you can request from the address listed

## Maintenance

The device is maintenance free.

## Changing the light source

- 🔧 The LED unit cannot be replaced. In case of a fault or defect, please contact HEINE.


## Replacing fuses

- ⚠ Switch off the examination light and remove the power cable from the device. The fuses are located in the fuse holder of the panel mount connector. Use only the fuses specified in the technical data as replacement fuses.

For repair, send the device to HEINE or an authorized service partner only, to avoid the risk of electrical shocks, burns or injuries due to improper repair work or handling.

## Technical safety inspection

- ⚠ The power supply unit and power cable should be inspected for damage and that they are in good working order at regular intervals. The technical safety inspection must be carried out every two years in accordance with the generally accepted rules of technology.

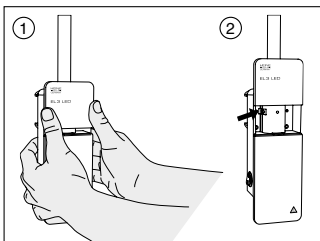- ① For the electrical inspection, the installation cover must be removed first.
- ② The testing point is located on the left hand side and is labelled accordingly.

## Service

The device has no components serviceable by the end-user.

## General Notes

- 🔧 The warranty for the entire product is invalidated if non-genuine HEINE products or non-original parts are used and if repairs or modifications are made to the device by persons not authorized by HEINE. For more information, please visit [www.heine.com](http://www.heine.com).

The expected life cycle, when the device is normal used and the warning and safety information as well as the maintenance instructions are observed, is up to 7 years. Beyond this period, the product may continue to be used if it is in a safe and good condition.

Condensation can form when moving from a cold to a warm environment. Wait until the unit has warmed up and is absolutely dry before using it.

## General Warnings

- ⚠ Check the correct operation of the device before use! Do not use the device if there are visible signs of damage.  
Do not modify the device.  
Use only original HEINE parts, spare parts, accessories and power sources.  
Repairs shall only be carried out by qualified persons.

## Disposal

- 🗑 The product must be recycled as separated electrical and electronic devices. Please observe the relevant state-specific disposal regulations.

The appendix contains following tables

- Electromagnetic disturbances – Requirements and tests
- Technical specification
- Explanation of the used symbols

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**Electromagnetic disturbances – Requirements and tests**

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should assure that it is used in such environments.

Statement for the operational environments	Inside hospitals except for: near active HF surgical equipment and the RF shielded room of an ME system for magnetic resonance imaging, where the intensity of EM disturbances are high.
Performance features of the ME system that have been determined to be essential to the performance	None
Warning	Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
	Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
	Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the EL3 LED Examination Light, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
A list of all cables, transducers and other accessories that are relevant for the EMC compliance	See attachment
<b>Test</b>	<b>Compliance</b>
RF emissions CISPR11	Group 1 Class B
Harmonic Emissions	Class A
Voltage Fluctuations/Flicker	Passed

## Technical specification

Mains power	100–240 V AC, 50/60 Hz
Power consumption	25 VA
Fuse sets	2 x T 2A H 250 V
Average LED operating life	typ. 30,000 hours
Minimum distance from light source to patient	20 cm
Working distance from light source to patient	30 cm
Central illuminance E <sub>c</sub>	≥ 42,000 lx*
Total irradiance E <sub>e</sub>	typ. 216 W/m <sup>2</sup> *
Colour temperature	4000 K ±400 K
Colour rendering index Ra	typ. 95
Special colour index R9	typ. 95
Environmental conditions for operation	+10 °C to +35 °C 30 % to 75 % rel. humidity 700 hPa to 1060 hPa
Environmental conditions for storage	+5 °C to +45 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Environmental conditions for transport	-20 °C to +50 °C 45 % to 80 % rel. humidity 500 hPa to 1060 hPa
Protection class	I
Operation altitude	≤ 3000 m
Protection provided by enclosures (IP code)	IP 20
Risk group (according to DIN EN 62471)	1 (low risk)
Weight incl. wall mount	approx. 1.8 kg
Weight of clamp mount	approx. 0.2 kg
Length of flexible arm	1 m

\*at a working distance of 30 cm, depending on t

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